



# INSTRUCTIONS

## AMERX<sup>®</sup> Wound Care Kit



### FOAM DRESSING

#### REMOVING THE DRESSING

Carefully remove the existing dressing/gauze that is covering the wound. To make the dressing easier to remove with little or no discomfort, you may dampen the dressing with AMERIGEL<sup>®</sup> Saline Wound Wash. Wait several minutes then remove carefully.

#### STEP 1: CLEANSING THE WOUND

Cleanse the wound with AMERIGEL Saline Wound Wash by spraying the affected area. The pressurized action of the Wound Wash requires only a small amount of spray to cleanse the wound. Gently blot the wound dry.




#### STEP 2: APPLYING THE AMERX FOAM DRESSING

Following wound cleansing, remove the AMERX Foam dressing from the package and place directly over the wound. The dressing should extend a minimum of ¼ inch beyond the wound borders. Carefully apply pressure to ensure proper contact.

#### STEP 3: APPLY THE AMERX BORDERED GAUZE DRESSING

Remove the AMERX Bordered Gauze dressing from the package. Remove one side of the paper liner. Place dressing over the wound and surrounding skin. Remove the remaining liner. Carefully smooth the edge of the dressing to ensure proper contact with surrounding skin.

**CHANGE THE DRESSING EVERY 3 DAYS OR AS INDICATED BY YOUR HEALTH CARE PROVIDER.**

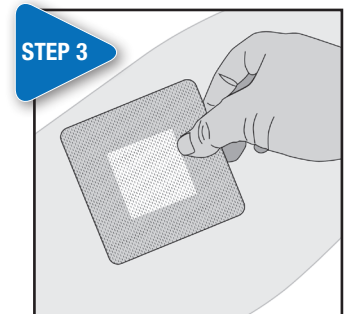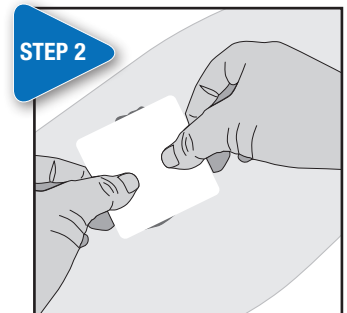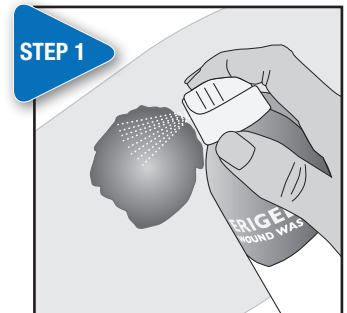

**FOR ANY QUESTIONS OR CONCERNS,  
PLEASE CONTACT YOUR PHYSICIAN**

**Attention:**  
Please place business card  
here before making copies.

**Your next scheduled visit:**

**Date:** \_\_\_\_\_

**Time:** \_\_\_\_\_ **AM / PM**